


Protective Sock

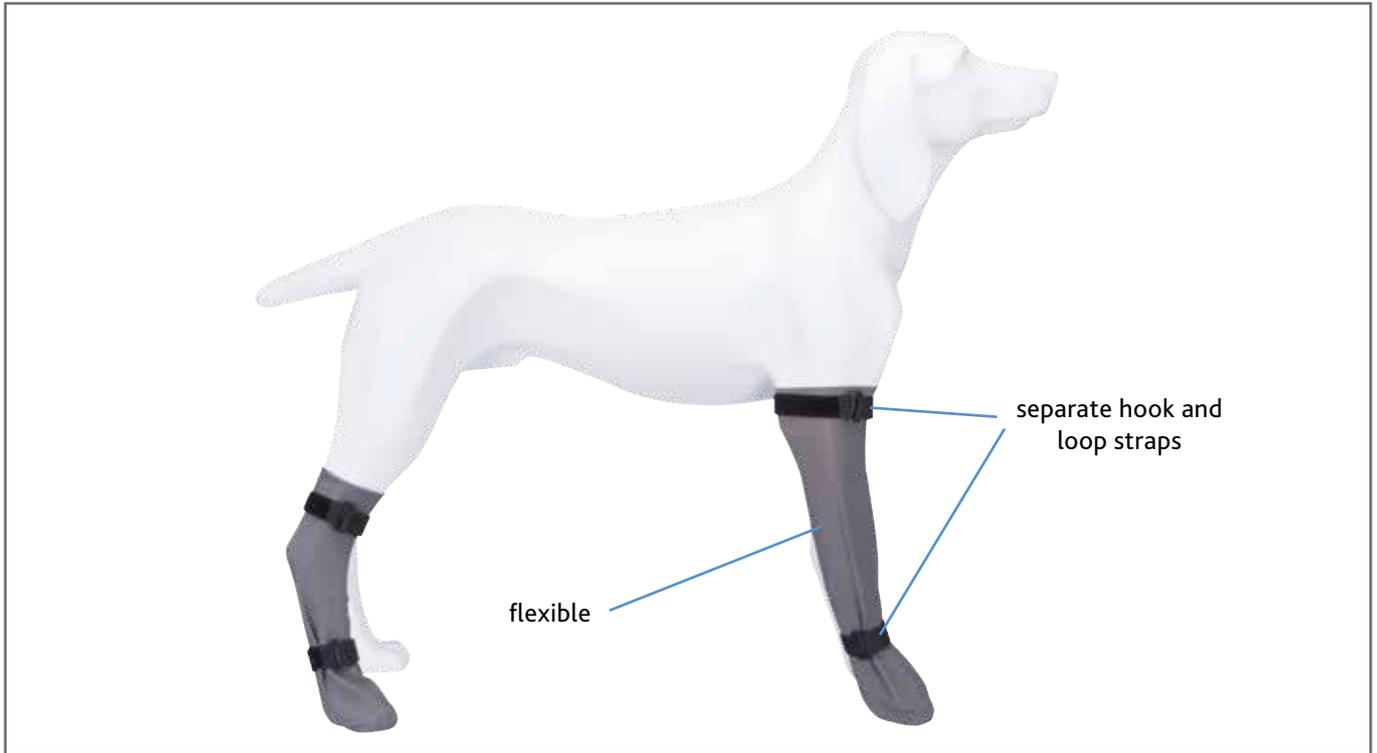

What is it for?

to keep wounds and bandages dry and clean, reduce the risk of infection and support the healing process

Product Information

- keeps full range of movement
- easy to fit by cutting
- #19431 also suitable for cats
- material: silicone, waterproof

Please Note

- How to determine the size: Measure the widest part of the paw while the dog is standing (taking bandages into account). The protective sock should be about 1 cm wider.
- Front leg: attach one fastener above the dew claw, the other one at the end of the sock. Hind leg: attach one fastener above the toe joint, the other at the end of the sock.
- Ensure a sufficient blood supply by not tightening the hook and loop fastener too tight.
- Not bite proof, only use when the dog is supervised.
- Air-tight, not for continuous use.
- No guarantee that the sock can withstand heavy use that it is not intended for (e.g. free playing, running on rough ground).

Item no.	Size	Width of the Sock	Inhalt	EAN
19431	S	6 cm/30 cm	1 pcs.	
19432	M	8 cm/35 cm	1 pcs.	
19433	L	10 cm/40 cm	1 pcs.	
19434	XL	12 cm/45 cm	1 pcs.	